**SUPRACOR'S SEATING & POSITIONING WHEELCHAIR CUSHIONS**  
**FEATURING STIMULITE® HONEYCOMB**  
Instructions for Use

# DIRECTORY OF LANGUAGES

|                  |    |
|------------------|----|
| English .....    | 3  |
| Dansk .....      | 9  |
| Deutsch .....    | 15 |
| Ελληνικά .....   | 21 |
| Français .....   | 28 |
| Íslenska .....   | 35 |
| Italiano .....   | 42 |
| Nederlands ..... | 49 |
| Norsk .....      | 56 |
| Suomi .....      | 62 |
| Svenska .....    | 68 |

**Supplier: Please give this manual to the user of this product.**

## TABLE OF CONTENTS

|                                         |   |
|-----------------------------------------|---|
| INTENDED USE.....                       | 4 |
| PRODUCT DESCRIPTION .....               | 4 |
| IMPORTANT SAFETY INFORMATION .....      | 5 |
| USING YOUR STIMULITE® CUSHION .....     | 5 |
| CARING FOR YOUR STIMULITE® CUSHION..... | 7 |
| WARRANTY INFORMATION .....              | 8 |
| RETURN POLICY .....                     | 8 |

Supracor, Inc., 2050 Corporate Court, San Jose, CA 95131-1753 U.S.A.

U.S.A.: 800.787.7226 • Fax: 888.329.7846 Customer Support: [orders@supracor.com](mailto:orders@supracor.com)

Outside the U.S.A.: 408.432.1616 • Fax: 408.432.8985

Made in U.S.A. ©2021 Supracor, Inc. All rights reserved. Supracor's products are covered under one or more of the following U.S. and corresponding foreign patents, as well as pending applications filed in the U.S. and abroad: 5,110,653; 5,122,405; 5,137,769; 5,180,619; 5,203,607; 5,289,878; 5,375,305; 5,403,642; 5,427,566; 5,444,881; 5,496,610; 5,509,484; 5,534,343; 5,617,595; 5,701,621; 5,756,182; 5,840,397; 5,840,400; 6,415,583; 7,376,996. Supracor, Stimulite and Corbee are registered trademarks and Slimline is a trademark of Supracor, Inc.

Product information is changed as needed; current product information is available at [Supracor.com](http://Supracor.com).

# STIMULITE® HONEYCOMB WHEELCHAIR CUSHIONS

## INTENDED USE

Stimulite Honeycomb Cushions are designed to relieve pressure and provide skin protection by contouring to each individual's anatomy while equalizing pressure across the cushion's surface.

## PRODUCT DESCRIPTION

Stimulite Honeycomb Cushions rely on a proprietary technology of fusion-bonded, flexible honeycomb which is anisotropic. The honeycomb is made from thermoplastic urethane that is naturally antimicrobial and allergen free, non-toxic and safe for the body and the environment. The cushion's cellular matrix consists of alternating thick-and thin-walled cells that flex when compressed to relieve pressure. Perforations in the cell walls allow air to circulate and moisture to evaporate, helping to keep the user cool and dry.

Each of the following products utilizes Stimulite flexible honeycomb technology to provide *pressure relief, skin protection, positioning and ventilation to help prevent pressure injuries among wheelchair users*:

Stimulite® Contoured, Contoured XS, Contoured Broader Base and Contoured Broader Base XS (Available in a flat or sling bottom)

Stimulite® Slimline™ and Slimline XS (Available in a flat or sling bottom)

Stimulite® Contoured Pediatric (Available in a flat bottom)

Stimulite® Adjustable Contoured and Adjustable Contoured XS (Available in a flat bottom)

Each of the following products utilizes Stimulite flexible honeycomb technology to provide *pressure relief, skin protection and ventilation to help prevent pressure injuries among wheelchair users*:

Stimulite® Classic and Classic XS (Available in a flat or sling bottom)

Stimulite® Corbee® Pediatric (Available in a flat or sling bottom)

Stimulite® Sport (Available in a flat bottom)

Stimulite® Smart (Available in a flat bottom)

Stimulite® Silver (Available in a flat bottom)

Stimulite® Bariatric (Available in a flat bottom)

A seating specialist, doctor or clinician experienced with wheelchair seating should be consulted to determine if the Stimulite cushion model is appropriate. The product's intended use statement alone should not be relied upon to make that determination.

## IMPORTANT SAFETY INFORMATION

### General Warning

Skin around the sitting surface should be monitored on a regular basis for redness or discomfort. If any changes are observed, discontinue using the cushion and consult your doctor or seating specialist.

### General Warning

Be sure to use the cushion with the cover provided.

Do not use the cushion on top of or beneath any other cushion or materials.

Do not use the cushion as a flotation device.

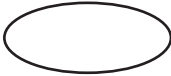
Do not expose the cushion to an open flame.

Do not expose the cushion to sharp objects.

## USING YOUR STIMULITE® CUSHION

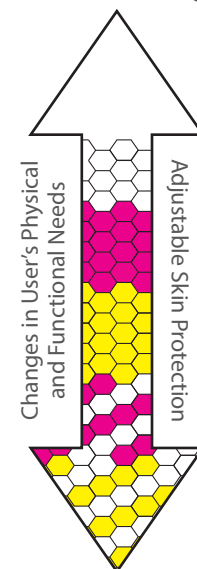
With the exception of the Stimulite Adjustable Contoured and Contoured XS Cushions, Stimulite cushions do not need any adjustment to provide pressure relief. They can be used on most wheelchairs. A sling bottom version of most of our Stimulite cushions is available for wheelchairs with a sling base.

The Stimulite Adjustable Contoured and Contoured XS Cushions feature an adjustment cavity at the bottom of the cushion that allows the user or caregiver to modify the cushion's firmness by selecting from multiple Stimulite honeycomb inserts that vary in softness. The softer insert allows for more immersion as compared to the firmer inserts. Removing the inserts all together provides additional relief to users with bony protuberances. In addition to adjusting the immersion, skin protection and positioning properties of the cushion, the bottom insert layer can also be used to address other positioning needs such as pelvic obliquities.

|                                                                                   | Insert           | Engineered specifically for:                                                                                                                                                                                                                                                                            |
|-----------------------------------------------------------------------------------|------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  | None             | A need to eliminate as much pressure as possible from the ischial tuberosities or other bony prominences.                                                                                                                                                                                               |
|  | White            | Hypotonicity or low tone, with accompanying high risk for skin breakdown. This hypotonicity can result in pelvic instability and subsequently poor sitting posture and positioning. Will offer increased pelvic engulfment.                                                                             |
|  | Magenta          | A need for less immersion (with maintenance of a moderate level of skin protection) as compared to incorporating no inserts as noted in row 1 of this table.                                                                                                                                            |
|  | Gold             | A need for a significantly stable base of support for the performance of transfers, but who presents with a reduced risk of skin breakdown (pressure injury) as compared to those who would be utilizing a white or magenta insert or no insert at all.                                                 |
|  | White on Magenta | A need for a significantly stable base of support for the performance of transfers, with an added layer of pressure relief.                                                                                                                                                                             |
|  | White on Gold    | A need for the highest level of support (horizontal stiffness) for stability (gold insert) with the highest degree of pressure relieving overlay (white insert), typically for a client who has not had a history of significant breakdown, as the suspension technique above is most helpful for this. |

For additional clinical guidelines for using Stimulite® inserts, visit: [www.supracor.com/products/show/166](http://www.supracor.com/products/show/166)

Deep Immersion  
with Positioning

Immersion with  
Added Support

### General Warning

A clinician or seating specialist should be consulted to determine specific positioning needs.

Stimulite cushions come with a breathable, removable cover that has a non-slip fabric and Velcro on the bottom side for securing the cushion to the wheelchair. Place the cushion on the wheelchair with the back of the cushion—where the zipper is located—firmly against the back of the wheelchair. Make sure that the cushion is firmly secured before sitting on the cushion.

### CARING FOR YOUR STIMULITE® CUSHION

Stimulite cushions and their covers are washable by hand or machine. The cushions come with a mesh laundry bag for machine washing. It is not necessary to use the laundry bag if washing the cushion by hand.

To wash your Stimulite cushion by machine, remove the cover and place the cushion in the laundry bag. Use a cold water setting and a small amount (1 TBSP) of mild laundry detergent. **DO NOT USE CHLORINE BLEACH WHEN WASHING THE CUSHION OR THE COVER.** The cover should be washed separately with cool to warm water and mild detergent. Press the Velcro together on the cover to prevent snags. Do not wash the cushion with other laundry since it may damage the honeycomb.

After washing, remove the cushion from the laundry bag, shake out any excess water and allow it to air dry. The cushion can also be placed in the dryer on a NO HEAT cycle. To expedite drying, place one or two dry bath towels in the dryer to absorb moisture. Do not dry the cushion with wet clothes as this causes heat retention which can damage the honeycomb.

After washing, place the cushion back in the cover with the curved back towards the zipper and the open honeycomb cells facing up. To ensure the open cells are facing up, there should be no labels on the top side of the honeycomb cushion. The labels are fused to the bottom side of the cushion.

Stimulite cushions are very durable and at the end of life they are recyclable. No adhesives or toxic materials are used in Stimulite cushions. If available, please contact an approved recycling site designated by your state or local authorities for recycling the cushion.

## WARRANTY INFORMATION

We offer a full two-year warranty on all of our Stimulite cushions. The warranty includes damage due to workmanship, material defects or errors caused by the manufacturer. The warranty does not include daily wear and tear or damage that is a result of disregarding the specific washing machine directions.

All of our cushions' covers are under warranty for 90 days. The warranty includes damage due to workmanship, material defects or errors caused by the manufacturer. The warranty does not include: cigarette burns, damage to the cover by sharp objects that may cause tears or color changes as a result of not washing the cover separately.

## RETURN POLICY

All returns require pre-authorization from Supracor's vendors or distributors and are subject to shipping and handling costs. If you have any questions about your cushion, please call us at 1.408.432.1616 or 1.800.787.7226 or visit our website at [www.supracor.com](http://www.supracor.com).

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